

PICTURE OF ULCER 2

ACCELERATED TISSUE REPAIR WITH ELECTRO-ACUSCOPE THERAPY

The following sequence of photographs illustrate the effectiveness of Electro-Acuscope/Myopulse therapy for wound healing applications.¹

DIAGNOSIS: Decubitus Ulcer

HISTORY: Two month old ulcer with no improvement

THERAPY: Electro-Acuscope treatments one per day from 5/15/90, through 5/22/90. Three times per week from 5/22/90, through 6/11/90. Treatment time was 20 minutes with two brass alloy plates placed on the wound perimeter. Instrument settings were 10 minutes at 160 Hz and 10 minutes at 80 Hz, with the Intensity set at 100 to 300 μ A.



5/22/90 - 6th treatment

5/29/90 - 8th treatment



6/11/90 - 15th treatment

¹ Photos courtesy of Christy Graves, M.D., Paula Simon, P.T., Director of Physical Therapy, Northshore Regional Medical Center in Slidell, Louisiana and Gwen Poulson, C.O.R.T., Director of Electrical Therapy at the clinic of Christy Graves, M.D.

